


## VERIclean®: Surface Hygiene Indicator



**VERIclean® Test Kit**

“In all cases, the results obtained were consistent and easy to interpret and were attainable within one minute of sampling the surface” [Figure 1]

- Ginny Moore and Chris Griffith, “Hygiene checks the next generation,” *International Food Hygiene*, vol.13, no.7

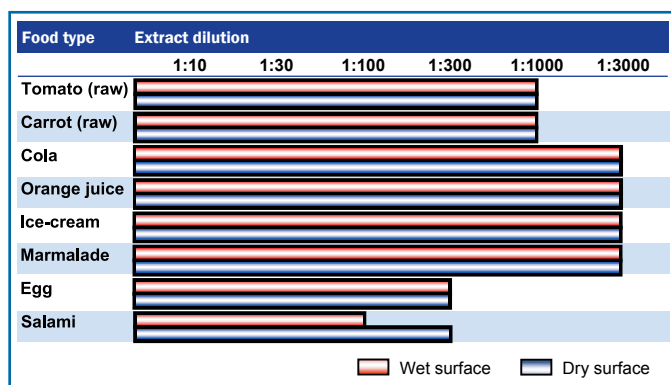

**Figure 1:** The detection of residual food debris from a wet and dry stainless steel surface, using the VERIclean surface test.

Source: Hygiene Checks-the next generation

### VERICLEEN AT A GLANCE

- ▶ Detects: Organic residue found on unknown surfaces and hand
- ▶ Method: Merely wet, wipe, and watch
- ▶ Economical: Needs no instrument
- ▶ Fast: 5 to 60 seconds
- ▶ Results: Accurate and repeatable

### SIMPLIFIED SURFACE MONITORING

VERIclean™ provides an inexpensive rapid hygiene test with many food/healthcare applications. VERIclean is simple to use, conveniently packaged and easy to interpret. There is no equipment, no special skills or training needed. VERIclean is a powerful tool for cleansing programs, hand washing, hygiene compliance, and overall sanitation monitoring.

### PROVIDING HYGIENE SAFETY

Organic residues can be nutrients for bacterial contamination, and they impede disinfectant and sanitizer effectiveness.

VERIclean validates the quality of cleaning of both product contact surfaces and employee hands by detecting organic material.

# VERicleen: Surface Hygiene Indicator

## RECOMMENDED FOR:

Airlines  
Auditors  
Beverage Processors  
Butcher Shops  
Caterers  
Cleaning Companies  
Commissaries  
Custom Blending  
Cruise Ships  
Dairy Producers  
Day Care Centers  
Delis  
Fast Food Chains  
Food Distributors  
Food Processors  
Health Inspectors  
Hospitals  
Hotels  
Nurseries  
Nursing Homes  
Restaurants  
Schools  
Supermarkets

## PROCEDURE

①




**Wet:** If surface is dry, wet with 3 or more pumps of Wetting Liquid. Enough liquid must be present on surface to absorb into test when surface is wiped

②



**Wipe:** Wipe surface with absorbent region of VERicleen. Check that Wetting Indicator is dark (indicating test strip is sufficiently wet).

③

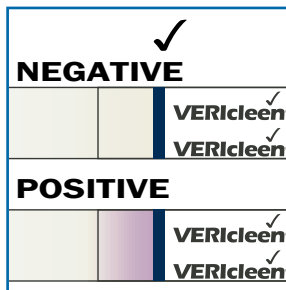

**Watch:** Purple color formation within 1 minute indicates presence of organic residues and need to re-clean.



Supplier Details:

### VERICLEEN: [CODE: VERICLEEN-50K]

- ▶ 50 VERicleen tests in a black snap cap container
- ▶ 1 bottle of Wetting Liquid with atomizer
- ▶ Instruction leaflet

### VERICLEEN: [CODE: VERICLEEN-100K]

- ▶ 100 VERicleen tests in a black snap cap container
- ▶ 2 bottles of Wetting Liquid with atomizer
- ▶ Instruction leaflet

## Charm Sciences, Inc.

659 Andover Street, Lawrence, MA 01843-1032, USA

Tel: +1.978.687.9200 | Fax: +1.978.687.9216 | Email: info@charm.com | www.charm.com

© Copyright 2010 Charm Sciences, Inc. Charm and VERicleen names are registered trademarks of Charm Sciences, Inc.

MRK-053-001 0810

